
Xal-Ease*

Delivery aid for administering XALATAN**
and XALACOM*** Ophthalmic Solutions

RECTO

PAA162203
146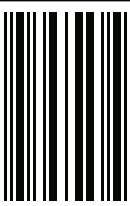

Please consult the XALATAN** and XALACOM*** instructions for further information on the use of your eyedrops.

INSTRUCTIONS FOR USE

XAL-EASE is an aid used for the delivery of XALATAN or XALACOM to the eye(s). The delivery aid holds a bottle of XALATAN or XALACOM and, when properly used, it will help you position the bottle over the eye to deliver a drop of the medication into the eye. A bottle cap opener is provided in the carton with the delivery aid. The bottle cap opener allows for removal of the cap from the bottle of eye drops after the bottle is placed in the delivery aid. The supplies you will need in order to use the delivery aid are shown in Figure 1.

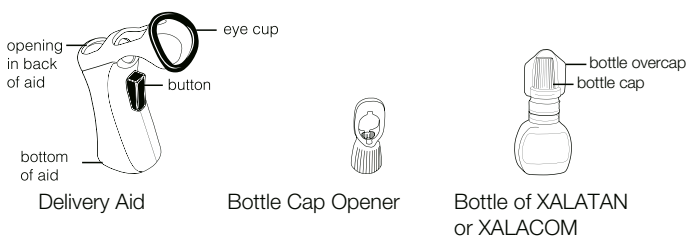
Figure 1

Before you administer the eye drops, wash your hands thoroughly with soap and water.

Preparing the aid for use

Each bottle of XALATAN or XALACOM has 2 caps as shown in Figure 1 - a winged overcap and a bottle cap. Remove only the overcap from the bottle of XALATAN or XALACOM by slightly twisting it with your fingers so the overcap breaks away from the bottle. Discard the overcap. Insert the bottle into the opening in the back of the delivery aid. Ensure the bottle is pushed fully into the delivery aid (Figures 2 and 3).

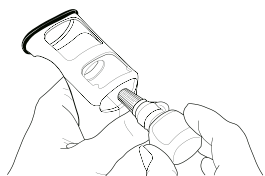
Figure 2

Figure 3

Turn the delivery aid so the cap on the bottle of eye drops is facing you. Remove the cap from the bottle by unscrewing it using the bottle cap opener (Figure 4). Alternatively, the cap can be removed with your fingers.

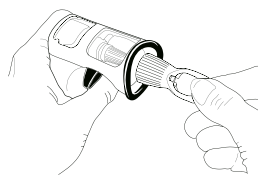
Figure 4

Placing the delivery aid over the eye area

Tilt your head back. Place the forefinger of your free hand at the top of your cheek and pull your lower eyelid down slightly. Using your other hand, place the eye cup of the delivery aid over the eye area so it rests on your skin. Be sure your head is tilted back far enough so that the aid is directly over your eye. This will allow for proper delivery of the drop of XALATAN or XALACOM (Figure 5).

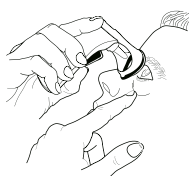
Figure 5

Administering the eye drops

When the eye cup is in place, look upward, and press the button until one drop is released from the bottle and enters the eye. Promptly remove your finger from the button to avoid the release of additional drops (Figure 6).

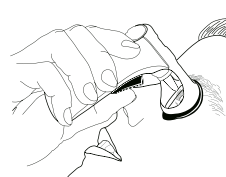
Figure 6

Handling and storing aid following use

After removing the delivery aid from the eye area, replace the bottle cap using the bottle cap opener or your fingers. The delivery aid should be placed on a firm surface with the XALATAN or XALACOM bottle in the upright position. Keep the delivery aid with the XALATAN or XALACOM bottle in the outer carton in order to protect from light. Consult XALATAN or XALACOM product information for proper storage conditions.

Changing to a new bottle of eye drops

Remove the empty bottle by pushing the cap end of the bottle with your thumb or finger so the bottle comes out through the opening in the back of the aid (Figure 7). Place a new bottle of XALATAN or XALACOM in the delivery aid as described above so it is ready to use for your next treatment.

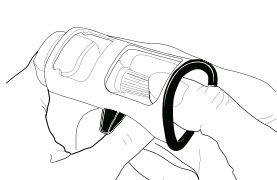
Figure 7

Cleaning the delivery aid

The delivery aid may be cleaned with mild soapy water and rinsed with warm tap water.

Replacing the delivery aid

The XAL-EASE delivery aid should last for approximately 3 years under normal conditions of use. However, it should be replaced if it is not working properly.

To request a replacement delivery aid, or for answers to questions about its use, please contact your physician.

Upjohn Canada ULC
Kirkland, Quebec H9J 2M5

*© Upjohn US 2 LLC, Upjohn Canada ULC, Licensee
**© Pfizer Health AB, Upjohn Canada ULC, Licensee
***© Pfizer Inc., Upjohn Canada ULC, Licensee

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* M.D. de Upjohn US 2 LLC, Upjohn Canada SRI, licencié
** M.D. de Pfizer Health AB, Upjohn Canada SRI, licencié
*** M.D. de Pfizer Inc., Upjohn Canada SRI, licencié